LM Global Design Limited

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GB Installation instructions Attach the 2 x L-shaped brackets to the KING raised toilet seat: turn the model upside down on a flat surface and insert the L-shaped brackets into the slots. Push gently on the bracket and turn the knob clockwise until the thread engages. Position the brackets leaving a greater width than that of the toilet bowl. Lift up existing toilet seat and place the KING seat directly onto the toilet bowl positioning the brackets towards the rear and the lip against the front of the bowl.

Turn the adjustment knobs clockwise until a resistance is felt and the seat does not move from side to side. Ensure seat is firmly fixed before using.

ATTENTION! The side-locks have been especially designed to allow for easy tightening therefore turning should not be continued once a resistance is felt.

Cleaning Instructions All parts can be cleaned with warm water and a neutral cleanser. Alternatively, the seat can be totally immersed in water and/or can be sanitized (without brackets) in an autoclave at 85°C for 1 minute.

Cautions Do not exceed maximum user weight: 225kg/496lb Always ensure the toilet seat is securely fixed before using. Do not hold onto seat for support while rising/lowering. KING bidet Portable bidet for use with KING raised toilet seats. Fits snugly allowing easy transferal. Soap dish located at the front to assist personal cleaning.

User Instructions Fill KING bidet with warm water and place inside installed KING raised toilet seat. Use as normal bidet. Tip: use water into toilet bowl.

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