

INTENDED USAGE:

The *KING Bathtub Mat* is meant to be used as a coating for the bottom of the bathtub. The rough surface increases the adherence with the floor, while the warm and soft material reduces the discomfort and insecurity due to the direct contact with the cold and smooth surfaces of the bathtub. The certified antibacterial properties of the composing material limit their proliferation, making the use suitable for humid and wet environments. The mats promote the user's independence while entering and exit the bathtub, making the support surfaces less slippery, facilitating its use in standing position.

INDICATIONS AND CONTRAINDICATIONS:

The KING Bathtub Mat is aimed to elderly people, people with reduced mobility and to all those who feel insecure while entering and exit the bathtub and using it in standing position. For a safe and autonomy use of the mats, the user is required to have sufficient ability to move and keep the standing position.

SAFETY WARNINGS AND INDICATIONS:

- If the user has mobility problems or suffers from balance disorders, we suggest to ask for assistance to enter and exit the bathtub.
- Do not modify the bathtub mat in any way.
- During the sitting and standing stages, the sliding risk may increase due to wet floor.
- The bathtub mat must not be stored outdoor.

MAINTENANCE INFO:

The product maintenance consists of the cleaning and sanitation procedures. In order to guarantee a safe use and adequate standard of hygiene, the user should perform these procedures before every use.

The user has to make certain of the structural integrity of the device. For cleaning and sanitation procedures, follow the steps below:

1. Gloves should be worn.
2. Remove the evident dirt before to proceed with the most accurate disinfection.
3. Use water or non aggressive detergent for the cleaning procedure.
4. Wipe with a dry and clean cloth.

The bathtub mat must be cleaned strictly by hand.

REUSE REQUIREMENTS:


The product is suitable for reuse by other users. Before reuse by others, the product must be correctly cleaned and sanitized. Before the transfer of the product to others, the product must be checked by qualified dealers.

CORRECT DISPOSAL PROCEDURE:


Do not dispose of the product in the normal household waste. Ask your local authority beforehand for the correct and environmentally friendly way to dispose it.

INCIDENT REPORTING:

If there was a serious incident involving this device, contact the manufacturer of this product at info@lmglobaldesign.com and the responsible authority in your country.

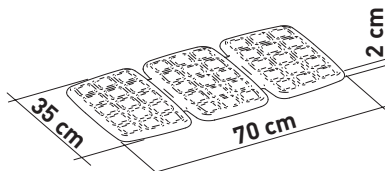

BATHTUB MAT

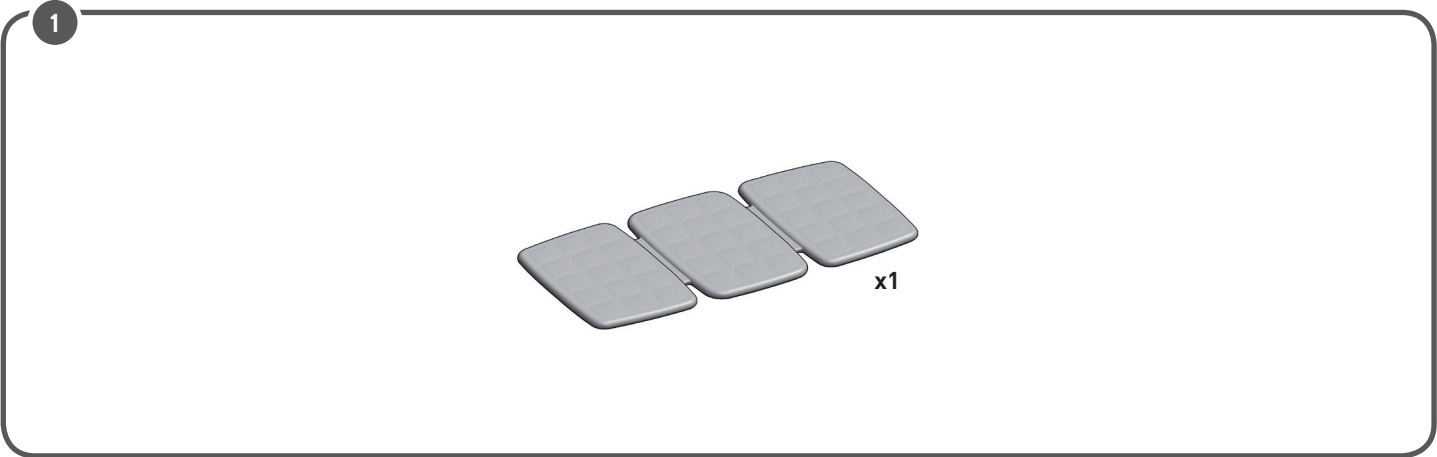

MADE IN ITALY



2 YEARS
GUARANTEE



LM Global Design Ltd.
Guinness Enterprise Centre,
Taylor's Lane, D8,
Dublin, Ireland

(t) +353 1 6874012
(w) www.lmglobaldesign.com
(e) info@lmglobaldesign.com